



Medical Pet Shirts is the founder and world leading company in covering and protective products for pets.



“With our passion for pets,
we love to make their
recovery as comfortable and
stress-free as possible.”

Nicole Kok

CEO, Medical Pet Shirts International

Contents

Page:

Introduction	2
MPS products	4
- Covering apparel	
- Medical Pet Shirt®	7
- TOP Shirt 4-in-1®	11
- TAZ - Front Leg Sleeves® (Single and Double)	14
- HLS - Hind Leg Sleeves®	18
- Comparison Chart	21
- Cooling apparel	
- DRY Cooling Vest®	22



MPS products:

100
percent
Original

100
percent
Patented

0
percent
Lead / Toxins

100
percent
**Animal
Friendly**



Introduction

Imagine this: a pet is recovering from skin conditions, hot spots or wounds (first aid, after being spayed, neutered or other surgical procedures). This can be quite stressful already. It would not benefit that pet if that discomfort and stress level is increased further by restricting their ability to move, eat or drink normally. This can happen when a 'cone of shame' (the old fashioned Elizabethan collar) is used.

Hearing clearly is very important for pets. Amplifying the sounds by using a cone can distract their sense of direction and could cause them to bump into something (valuable furniture for example).

The best way

Here at MPS, we believe it is better to cover the problem area instead of restricting the head to prevent dogs, cats and rabbits from getting to their surgical sutures, wounds, hot spots or skin problems.

Children friendly

Pets are protected from themselves as well as from other animals/children.

Easy to use, Patented Designs


MPS products act as a second skin, allowing pets to enjoy their day with little anxiety or disruption.

Only the best, non-toxic materials

When protecting wounds, hot spots or skin conditions it is important to keep it dry and clean. To achieve this, we use the best high quality cottons (94 – 96 percent) to make our products as breathable and lightweight as possible without compromising strength and comfort. For stretch we use the highest quality Invista® Lycra.

**“Reduce stress and discomfort;
this helps for a quicker recovery”**



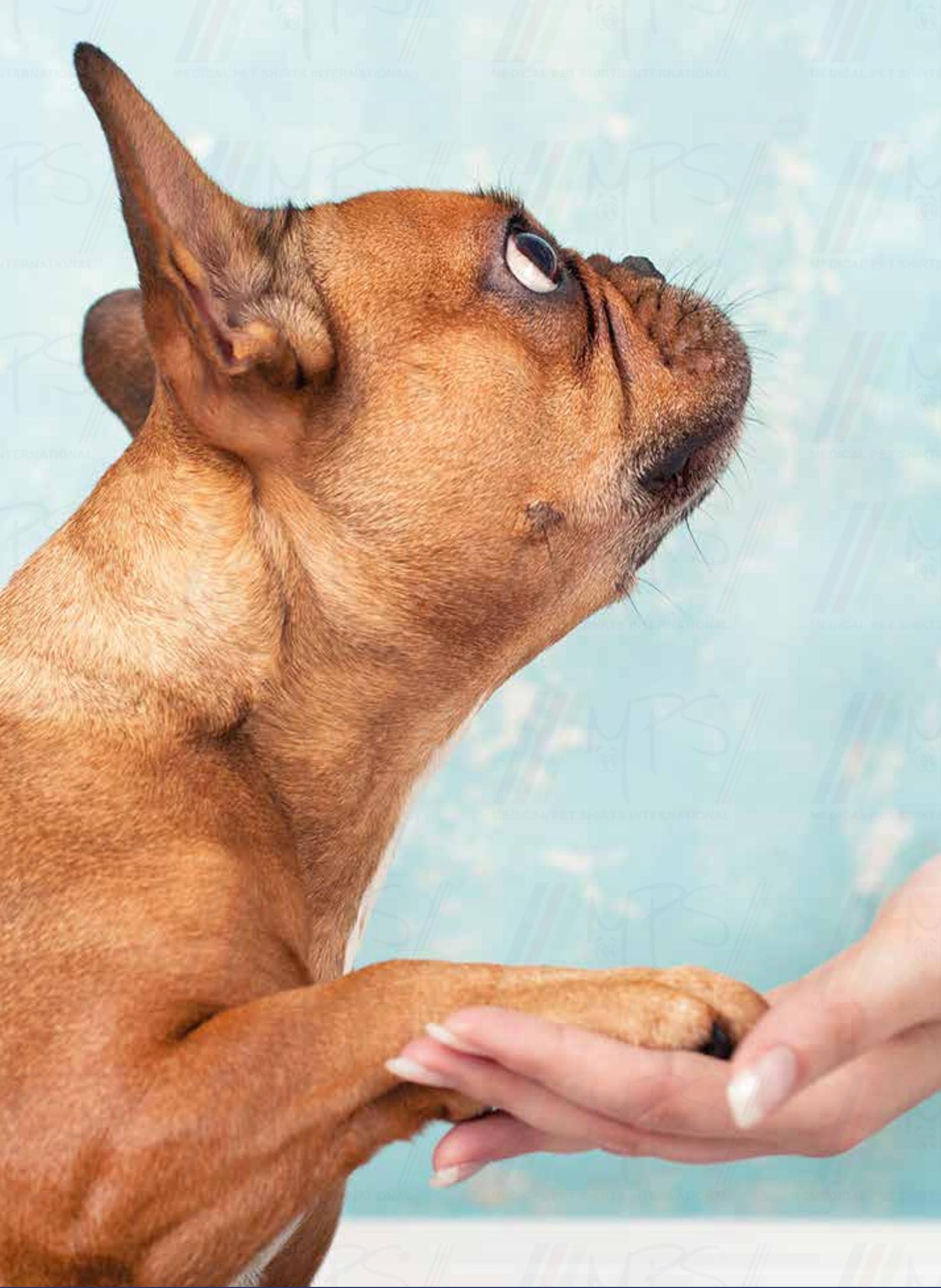

MPS - products



Medical Pet Shirt®

The Medical Pet Shirt for Dogs is a professional alternative to a cone and full body bandage. Suitable for first aid purposes, to protect and cover hot spots, wounds, surgical sutures and bandages, ointments, skin conditions, when in season or incontinence and a lot more...

The design is based on the animal's shape and anatomy with a snug fit as result. Manufactured using a breathable stretchy Cotton / Lycra fabric, giving complete freedom of movement without compromising strength or durability. Thanks to the rear closure effortless to fit.

The Medical Pet Shirt for Cats and Rabbits is different in size and shape compared to the Shirt for Dogs. For most cats and rabbits, it can remain fastened when nature calls, due to the specially tailored space under the tail.



TOP - Shirt "4-in-1"®

An ideal solution for wound protection providing comfort after surgery whilst allowing freedom of movement.

Manufactured using a soft double layered Cotton fabric around the body. The front side of the body is manufactured using a breathable stretchy Cotton / Lycra fabric with reinforced Cotton bands. Thanks to the adjustable closure around the chest fitting is effortless.





TAZ - Front Leg Sleeve (Single and Double)®



As in our Medical Pet Shirts, the Front Leg Sleeve is a left / right interchangeable front leg protection for recovering dogs. To cover hot spots, wounds, surgical sutures, IV lines and bandages, ointments, skin conditions and a lot more. Made from high quality cotton / stretch fabric to provide the best comfort possible. Thanks to the adjustable closure around the chest they are effortless to fit. This also applies some pressure around the chest with a snug fit as a result, which helps to reduce the stress level. The Double Front Leg Sleeve offers protection for both legs at the same time and also includes soft elbow pads.

HLS - Hind Leg Protection®

Wound and bandage protection and in cases of skin diseases whilst allowing freedom of movement.

Supplied as two sleeves left and right secured together with adjustable Velcro® and elastic straps to aid fitting. To ensure placement both hind leg sleeves need to be used, simply connect the integrated and adjustable elastic straps to the rear side loops of the MPS-TOP shirt, purchased separately.

The soft double layered fabric at the hips, back and front side of the sleeve is manufactured using Cotton. The rear side of sleeve is manufactured using a breathable stretchy Cotton / Lycra fabric.





DRY Cooling Vest®

Keep dogs COOL by using this comfortable and easy to refill DRY Cooling Vest. After activating the vest by filling it with fresh water, it will immediately start cooling up to 15°C / 59°F below ambient air temperature for one to three days. Thanks to the patented system, the vest will never become wet or damp during use. The cooling effect is the result of evaporation and is largely dependent on humidity and airflow.


So, prevent dogs from developing heat stress! During and after a walk (in the sun), in the car, or whenever cooling is required.



Cooling technology by:



DRY
EVAPORATIVE
COOLING

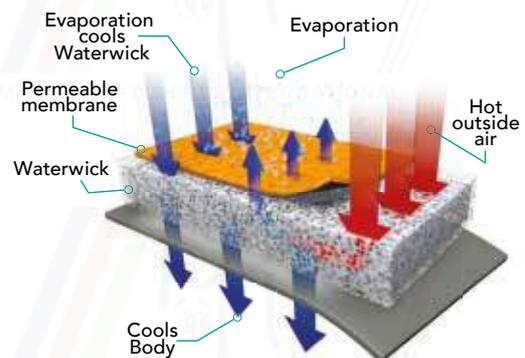


Medical Pet Shirt®

Multi-Purpose core body protection




- Allows pet to move freely in their environment
- Pets can eat and drink as normal
- Pets are protected from themselves as well as from others
- Breathable lightweight four-way stretch fabric
- Stress free design enables faster healing

- Multi-purpose use
- Machine washable
- Allows use of litter box for cats
- Perfectly snug fit
- Exclusive, patented design

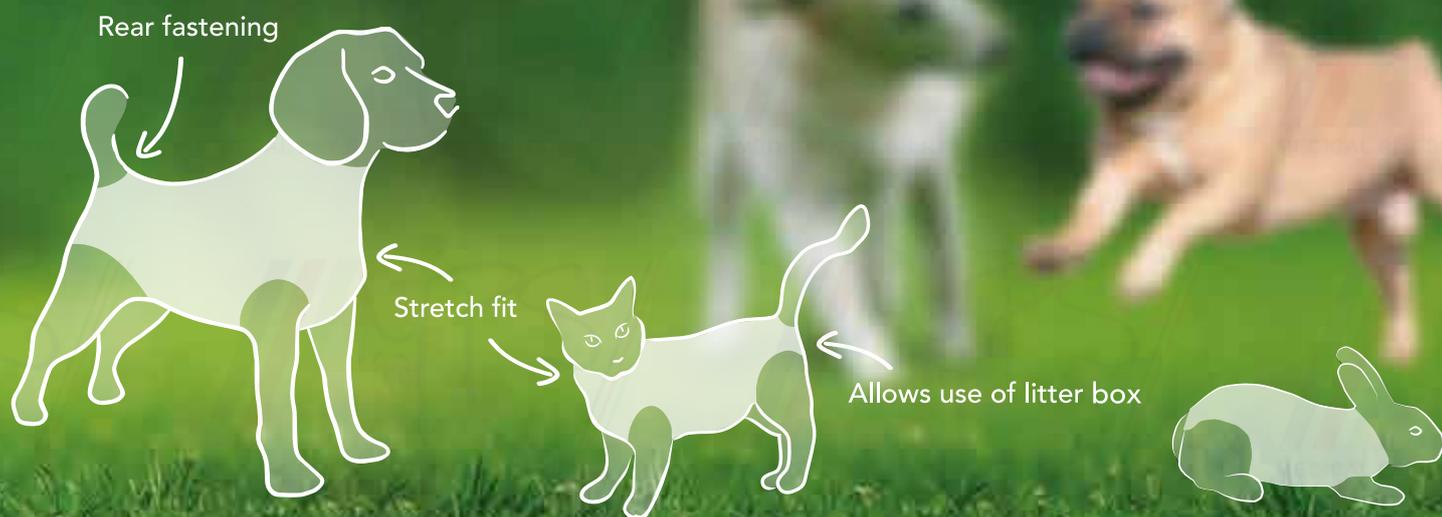



Organic Cotton

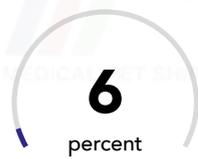

**Invista Lycra
(stretch)**



Lead / Toxins



Animal Friendly

“Comfortable recovery without any compromises”

The Medical Pet Shirt is a professional alternative to a cone and full body bandage. Suitable for **first aid** purposes, to cover and protect **hot spots, wounds, surgical sutures and bandages, ointments, skin conditions**, when **in season** or **incontinence** and a lot more...

The design is based on the animal's shape and anatomy with a snug fit as result. Manufactured using a breathable stretchy Cotton/ Lycra fabric, giving complete freedom of movement without compromising strength or durability. Thanks to the rear closure effortless to fit.

Allows use of litter box and outside

When walking a dog, only the rear end of the shirt needs to be opened, simply roll the underside of the shirt inwards and towards the belly and secure with the strap. For most cats, the shirt can remain fastened when using a litter box, due to the specially tailored space under the tail.



First Aid, Hot Spot, Wound, Suture and Skin protection

- Reduces the annoyance and stresses a pet may experience, compared to wearing an Elizabethan collar.
- Pets are protected from themselves as well as from others.
- Our exclusive breathable design enables air to circulate around the affected area while still keeping it dry.
- Helps keep patients warm after (and during) surgery.

The inner panel of the shirts is light colored allowing any fluid discharge to be easily seen. If the wound leaks, a porous sheet or incontinence pad can be inserted inside the Medical Pet Shirt or between inner panel.

When in season or incontinence

The Medical Pet Shirt reduces the risk of blood or urine stains in households. In contrast to standard shorts, it is very difficult for a pet to wriggle out of.


Skin conditions

The Medical Pet Shirt offers protection where it is required. It helps prevent possible damage to the skin caused by scratching or biting.

Sizes

Medical Pet Shirt – DOG

Size	Length <small>(from base of neck to base of the tail.)</small>
4X Small	19 - 30 cm / 7.5 - 11.8 inch
3X Small:	22 - 35 cm / 8.7 - 13.8 inch
2X Small:	33 - 42 cm / 13.0 - 16.5 inch
X Small: *	40 - 45 cm / 15.7 - 17.7 inch
Small: *	43 - 51 cm / 16.9 - 20.1 inch
Small + (plus):	49 - 57 cm / 19.3 - 22.4 inch
Medium: *	55 - 69 cm / 21.7 - 27.2 inch
Medium + (plus):	61 - 72 cm / 24.0 - 28.3 inch
Large: *	67 - 76 cm / 26.4 - 29.9 inch
X Large: *	74 - 82 cm / 29.1 - 32.3 inch
XX Large:	80 - 92 cm / 31.5 - 36.2 inch

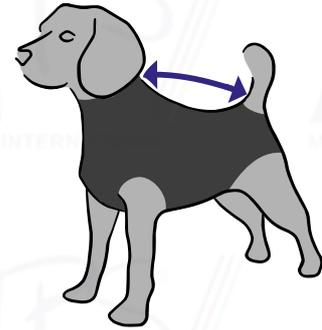


Available in:

-  Blue
-  Zebra print

Medical Pet Shirt - CAT

Size	Length <small>(from base of neck to base of the tail.)</small>
4X Small	25 - 32 cm / 9.9 - 12.6 inch
3X Small:	29 - 37 cm / 11.4 - 14.6 inch
2X Small: *	33 - 42 cm / 13.0 - 16.5 inch
X Small: *	40 - 45 cm / 15.7 - 17.7 inch
Small:	43 - 51 cm / 16.9 - 20.1 inch

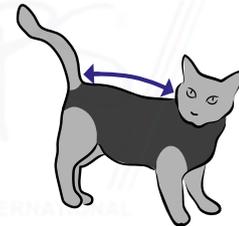


Available in:

-  Red
-  Zebra print
-  Leopard print

Medical Pet Shirt - RABBIT

Size	Weight
3X Small	0,6 - 0,8 kg / 1.3 - 1.8 lbs
2X Small:	0,8 - 1,5 kg / 1.8 - 3.3 lbs
X Small: *	1,5 - 2,0 kg / 3.3 - 4.4 lbs
Small: *	2,0 - 3,5 kg / 4.4 - 7.7 lbs
Medium:	3,5 - 4,5 kg / 7.7 - 9.9 lbs
Large:	4,5 - 5,5 kg / 9.9 - 12.1 lbs

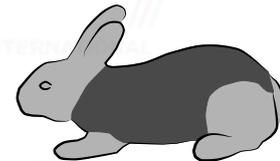


Available in:

-  Light green

* Most popular sizes

How to use

How to use:

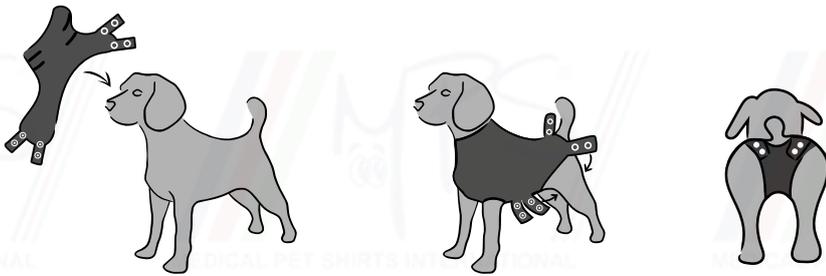

Step 1:

With large dogs (long legs) it is easier to insert one front leg followed by the head and then the other leg into the provided openings. With small dogs, cats and rabbits the head must be inserted first into the shirt followed by each front leg (separately) into the openings provided.

Step 2:

The Medical Pet Shirt should then be pulled back as far as possible over the pet's body.

Step 3:

The two straps on the belly side are secured by threading them between the rear legs and then attaching them to the press buttons at the base of the tail. Make sure that there is enough room for tail movement. If needed, secure the loops inside the shirt to the collar for extra security.

When nature calls:

The Medical Pet Shirt is designed for many purposes. It needs to be fully closed when used for light incontinence or after being spayed/neutered. When nature calls, the rear end of the shirt can be opened, simply roll the bottom part of the shirt inwards and towards the belly and secure with the added hold-up strap.

TOP-Shirt 4-in-1[®]

Not JUST protection, but a lot more...



- Protection of wounds, bandages and in cases of skin diseases
- Anxiety management
- Device holder
- Fixation of MPS-HLS Hind Leg Sleeves





Organic Cotton



**Invista Lycra
(stretch)**



Lead / Toxins



Animal Friendly

“Reduce stress, this helps for a quicker recovery”

An ideal solution for wound protection providing comfort after surgery whilst allowing freedom of movement. Manufactured using a soft double layered Cotton fabric around the body. The front side of the body is manufactured using a breathable stretchy Cotton / Lycra fabric with reinforced Cotton bands. Thanks to the adjustable closure around the chest fitting is effortless.

Protection of wounds, bandages and in cases of skin diseases

- Protects wounds and bandages specifically around the chest, shoulders and neck.
- Open hole at shoulders may be used to pass an IV-Line.
- Minimises injury to the skin due to scratching, licking or biting.
- Promotes faster healing in cases of skin conditions, medication cannot be licked off of the skin.

Anxiety management

Offers a gentle pressure around the chest thanks to the adjustable closure with elasticated lower band. This pressure has a calming effect for most animals especially useful for procedures such as knee surgeries when combined with the MPS-HLS Hind Leg Sleeves.

Also helps to reduce issues with barking, travel, fireworks or separation without the use of sedatives.

Device holder

Closable pocket on back (size 12(l) x 9(w) x 3(h) cm / 4.7(l) x 3.5(w) x 1.2(h) inch) to hold devices such as portable IV, Holter-ECG or suction pump.

Fixation of MPS-HLS Hind Leg Sleeves


Keeps hind leg sleeves firm in place with the integrated loops on the rear side.



Sizes and how to use

TOP - Shirt 4-in-1


Size	Chest circumference
TOP-2XS:	28 - 43 cm / 11 - 17 inch
TOP-XS:	38 - 52 cm / 15 - 22 inch
TOP-S:	48 - 68 cm / 19 - 27 inch
TOP-M:	60 - 80 cm / 24 - 31 inch
TOP-L:	72 - 92 cm / 28 - 36 inch
TOP-XL:	78 - 100 cm / 31 - 39 inch

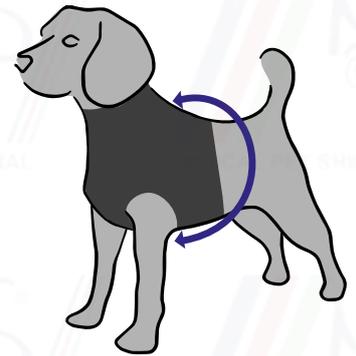

Available in:

-  Blue
-  Zebra print

How to use:

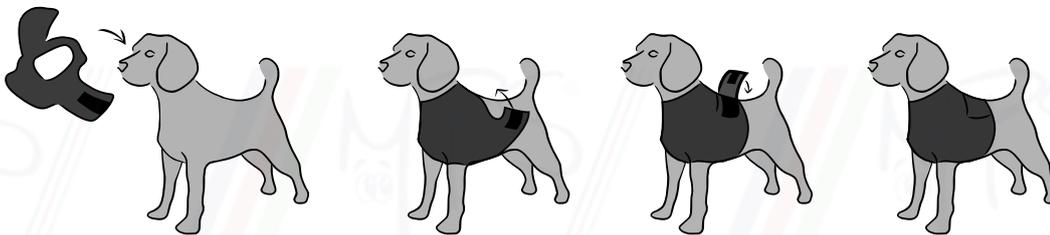

Step 1:

Put the MPS-TOP Shirt on by inserting the head first followed by each front leg separately into the openings provided.

Step 2:

Close the MPS-TOP Shirt around the chest with the adjustable Velcro.

Step 3:

If you want a more calming effect slightly increase the pressure.

* Most popular sizes

MPS-TAZ Front Leg Sleeve®

Single and Double

Multi-Purpose front leg protection



- Perfectly snug fit
- Exclusive, patented design
- Freedom to move as normal
- Easy to use
- Adjustable



Left / Right exchangeable



Extra elbow pads
(double sleeve only)



Organic Cotton



Invista Lycra
(stretch)



Lead / Toxins



Animal Friendly

“Reduce stress, this helps for a quicker recovery”

The TAZ - Front Leg Sleeve is an interchangeable front leg protection for recovering dogs. Suitable to cover and protect **hot spots, wounds, surgical sutures and bandages, ointments, skin conditions, IV-lines** and a lot more. Made from high quality cotton / stretch fabric to provide the best comfort. Thanks to the adjustable closure around the chest they are effortless to fit. This also applies some pressure around the chest with a snug fit as a result, which helps to reduce the stress level.


The Double Front Leg Sleeve offers protection for both legs at the same time. It also includes an elbow pad on each leg.



Sizes and how to use

TAZ - Single Front Leg Sleeve


Size	Chest circumference	Length Sleeve
XXX Small:	27 - 33 cm / 10.6 - 13.0 inch	10 cm / 3.9 inch
XX Small:	30 - 43 cm / 11.8 - 16.9 inch	12 cm / 4.7 inch
X Small: *	40 - 52 cm / 15.7 - 20.5 inch	17 cm / 6.7 inch
Small: *	45 - 61 cm / 17.7 - 24.0 inch	23 cm / 9.1 inch
Medium: *	54 - 68 cm / 21.3 - 26.8 inch	26 cm / 10.2 inch
Large: *	64 - 84 cm / 25.2 - 33.1 inch	35 cm / 13.8 inch
X Large: *	80 - 104 cm / 31.5 - 40.9 inch	41 cm / 16.1 inch
XX Large:	96 - 124 cm / 37.8 - 48.8 inch	48 cm / 18.9 inch

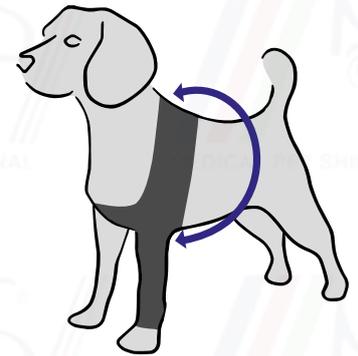

Available in:



How to use (Single Sleeve):

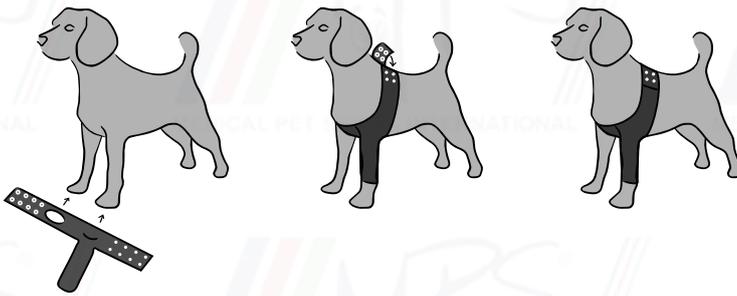

Step 1:

Insert one leg through the sleeve;

Step 2:

Insert the other leg through the opening on the opposite;

Step 3:

Secure around the chest by using the snaps/press buttons. (Always secure with at least four snaps)

* Most popular sizes

Sizes and how to use

TAZ - Double Front Leg Sleeve


Size	Chest circumference	Length Sleeves
XXX Small:	27 - 33 cm / 10.6 - 13.0 inch	10 cm / 3.9 inch
XX Small:	30 - 43 cm / 11.8 - 16.9 inch	12 cm / 4.7 inch
X Small: *	40 - 52 cm / 15.7 - 20.5 inch	17 cm / 6.7 inch
Small: *	45 - 61 cm / 17.7 - 24.0 inch	23 cm / 9.1 inch
Medium: *	54 - 68 cm / 21.3 - 26.8 inch	26 cm / 10.2 inch
Large: *	64 - 84 cm / 25.2 - 33.1 inch	35 cm / 13.8 inch
X Large: *	80 - 104 cm / 31.5 - 40.9 inch	41 cm / 16.1 inch
XX Large:	96 - 124 cm / 37.8 - 48.8 inch	48 cm / 18.9 inch



Available in:



How to use (Double Sleeve):

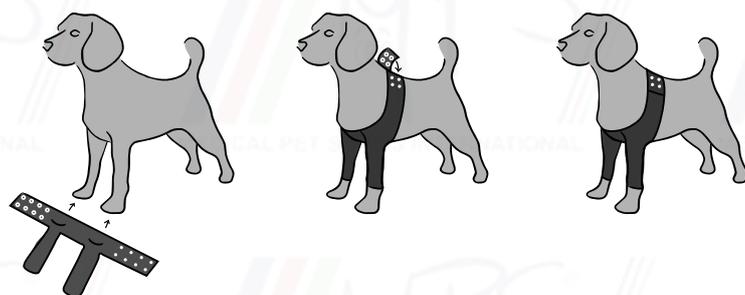

Step 1:

Insert both legs through the sleeves;

Step 2:

Secure around the chest by using the snaps/press buttons. (Always secure with at least four snaps)

* Most popular sizes

HLS - Hind Leg Sleeves[®]



- Protection of wounds, bandages and in cases of skin diseases
- Anxiety management
- Device holder
- Fixation of MPS-HLS Hind Leg Sleeves





Organic Cotton



**Invista Lycra
(stretch)**



Lead / Toxins



Animal Friendly

MPS-HLS Hind Leg Sleeves can be used for wound and bandage protection whilst allowing freedom of movement.

Supplied as two sleeves left and right secured together with adjustable Velcro® and elastic straps to aid fitting. To ensure placement both hind leg sleeves need to be used, simply connect the integrated and adjustable elastic straps to the rear side loops of the MPS-TOP shirt, purchased separately. The soft double layered fabric at the hips, back and front side of sleeve is manufactured using Cotton. The rear side of sleeve is manufactured using a breathable stretchy Cotton / Lycra fabric.

Applications

- The MPS-HLS Hind Leg Sleeves protects wounds and bandages at the hind leg.
- In cases of skin diseases it minimises injury to the skin due to scratching, licking or biting, of course always after consulting your veterinarian.
- It promotes faster healing because the medication can not be licked off the skin.

Benefits

- Protection of wounds and bandages
- Protection of skin diseases at the hind leg
- The MPS-TOP Shirt and the MPS-HLS keeps the dog warm after surgery
- The dog can keep the MPS-TOP Shirt and the MPS-HLS on while going outside
- The MPS-TOP Shirt and MPS-HLS are comfortable, hygienic and machine washable at 30C.

Usage

The MPS-HLS can only be used in combination with the MPS-TOP Shirt. The MPS-TOP Shirt has integrated loops on the rear side to attach the MPS-HLS.

Post Operative

The MPS-HLS can only be used in combination with the MPS-TOP Shirt. This combination reduces stress, anxiety and fear after surgery. It promotes faster healing because medicine can't be licked off the skin.

As it is not a tight fit, air is able to circulate around the wound serving to keep the area dry.

The combination can be used in cases of skin disease.


The MPS-HLS Hind Leg Sleeves minimise injury to the skin due to scratching, licking or biting.



Sizes and how to use

MPS - HLS Hind Leg Sleeves

Size	Length Sleeves
HLS-2XS:	10 cm / 4.5 inch
HLS-XS:	14 cm / 5.5 inch
HLS-S:	20 cm / 8.0 inch
HLS-M:	26 cm / 10.0 inch
HLS-L:	32 cm / 12.5 inch
HLS-XL:	37 cm / 14.5 inch

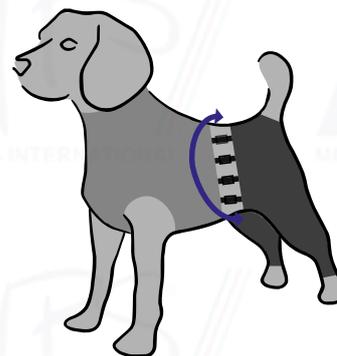

Available in:

 Blue

How to use:

Step 1:

Put the MPS-TOP Shirt on by inserting the head first followed by each front leg separately into the openings provided.

Step 2:

Close the MPS-TOP Shirt around the chest with the adjustable Velcro. If you want a more calming effect slightly increase the pressure.



Step 3:

Separate the Hind Leg Sleeves. Put the operated leg in the first sleeve and attach the elastic straps to the integrated loops of the MPS-TOP Shirt. Secure the straps firmly.

Step 4:

Put the other leg in the second sleeve and attach both sleeves to each other with the Velcro closure on the back part.

Attach the elastic straps firmly to the loops on the MPS-TOP Shirt.

On the back part attach them in a straight line. Attach the side straps preferably upwards towards the back.

Step 5:

Link the sleeves together with the straps under the tail.

Secure firmly to prevent movement of the sleeves.

Make sure the MPS-TOP Shirt and the Hind Leg Sleeves are properly secured.

* Most popular sizes

Comparison chart

**"Reduce stress and discomfort;
this helps for a quicker recovery"**



	Product				
	Medical Pet Shirt	TAZ - Front leg Sleeve	HLS - Hind leg Sleeve	TOP-Shirt 4-in-1	Traditional E-Collar
Covers the affected area	✓	✓	✓	✓	
Restricts the head					✓
First Aid	✓	✓	✓	✓	
Anxiety management	✓	✓	✓	✓	
Protects:	✓	✓	✓	✓	
- Hot spot	✓	✓	✓	✓	
- Wound	✓	✓	✓	✓	
- Bandage / IV line	✓	✓	✓	✓	
After surgery:	✓	✓	✓	✓	✓
- When Spayed / Neutered	✓				
- Wound management	✓	✓	✓	✓	✓
Ointment / salve protection	✓	✓	✓		
Allergies / Skin conditions	✓	✓	✓	✓	✓ head only
Incontinence / when in season	✓				
Safe around children	✓	✓	✓	✓	
Safe around pet's best friends	✓	✓	✓	✓	
Puppy/Kitten nursing control	✓				
Comfortable	✓	✓	✓	✓	
Reduces stress	✓	✓	✓	✓	
Freedom to move, eat and drink	✓	✓	✓	✓	
Furniture friendly	✓	✓	✓	✓	

Used for



DRY Cooling Vest[®]

"The ultimate heat-stress prevention!,,



50

percent

Exterior:

50% Polyester Jersey,
50% PU Ultra Fresh
Antibacterial

100

percent

Core:

Inuteq™
technology
Wicking Felt

50

percent

Interior:

50% Nylon,
50% PU

0

percent

Lead / Toxins

100

percent


Animal Friendly

“Keep your dog COOL and DRY, this helps to avoid heat stress”

Keep dogs COOL by using this comfortable and easy to refill DRY Cooling Vest. After activating the vest by filling it with fresh water, it will immediately start cooling up to 15°C / 59°F below ambient air temperature for one to three days. Thanks to the patented system, the vest will never become wet or damp during use. The cooling effect is the result of evaporation and is largely dependent on humidity and airflow.

So, prevent dogs from developing heat stress! During and after a walk (in the sun) or whenever cooling is required.

- ❄️ Comfortable Heat-Stress Management;
- ❄️ Cools down up to 15 C / 59 F below ambient air temperature;
- ❄️ Immediate cooling up to 3 days, 100% DRY inside and out;
- ❄️ Unique and patented DRY evaporation cooling technology;
- ❄️ Proven technology by sporters, military and professionals;
- ❄️ No refrigeration or additional gels or chemicals are required;
- ❄️ Antibacterial, Machine washable and easy to clean.

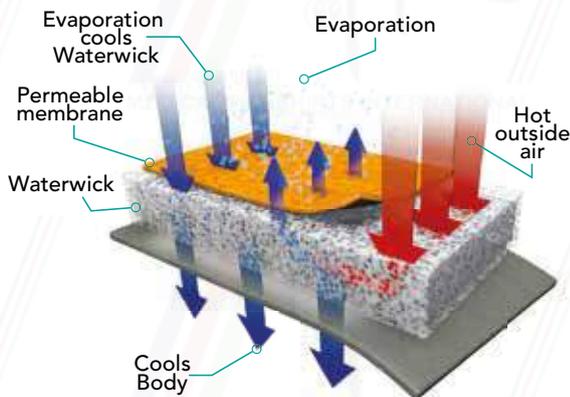

How does it work:


1. Hot outside air vaporizes water;
2. Evaporations extracts heat;
3. Surface cools down;
4. Fabric cools the body.

Cooling technology by:



DRY
EVAPORATIVE
COOLING

Sizes and how to use

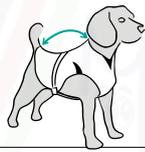

Size	Length <small>(from base of neck to base of the tail.)</small>	Chest circumference	Neck circumference	Water required
XSmall:	25 - 35 cm / 10.0 - 14.0 inch	40 - 60 cm / 16.0 - 24.0 inch	< 35 cm / < 14 inch	100 ml / 3.40 oz.
Small:	35 - 50 cm / 14.0 - 20.0 inch	55 - 70 cm / 22.0 - 28.0 inch	< 45 cm / < 18 inch	200 ml / 6.75 oz.
Medium:	50 - 60 cm / 20.0 - 24.0 inch	60 - 80 cm / 24.0 - 32.0 inch	< 50 cm / < 20 inch	290 ml / 10.20 oz.
Large:	60 - 75 cm / 24.0 - 30.0 inch	65 - 85 cm / 26.0 - 34.0 inch	< 55 cm / < 22 inch	430 ml / 14.50 oz.
XLarge:	70 - 90 cm / 28.0 - 36.0 inch	75 - 95 cm / 30.0 - 38.0 inch	< 65 cm / < 26 inch	525 ml / 17.70 oz.

Available in: ● Silver

To activate the DRY® Cooling Technology:



Fill with fresh water



Distribute water evenly.




Remove excess water and air then close the valve.



Vest starts to cool immediately (15 °C / 59 °F) for up to three days.

How to use:

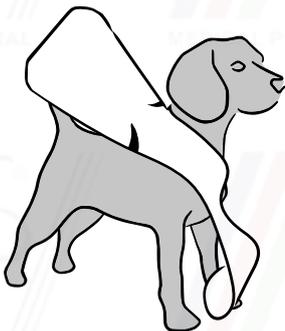

Step 1:

Insert the head through the opening;

Step 2:

Secure the vest by closing the buckles from the belly part to the back;

Step 3:

Adjust the straps to the right length;

Step 4:

Enjoy Immediate cooling.

DRY Cooling Vest





Nicole Kok
CEO

+31 (0) 79 36 25 668
nicole@medicalpetshirts.com
www.medicalpetshirts.com

LeAnn Wagner
United States Representative

+1 (219) 861 2009
LeAnn@medicalpetshirts.com
www.medicalpetshirts.com

Visit
Medicalpetshirts.com

